

CALCIPAST

Calcium hydroxide paste

CE₁₀₁₁

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY

COMPOSITION: calcium hydroxide, barium sulphate, hydroxyapatite, propylene glycol.

INTENDED USE

Product CALCIPAST is intended for use during dental treatment as material for temporary root canal fillings. Additionally, CALCIPAST may be used for temporary fillings of deep cavities during indirect or direct pulp capping. It has remineralising effect on tissues of tooth thanks to its high content of calcium hydroxide. Product improves rebuilding of damaged periapical tissues. pH≈12,5

INSTRUCTIONS FOR USE

Place attached applicator onto the syringe containing product.

In the root canal: insert the paste into the canal, remove the excess of the paste from the canal, close cavity securely.

For the cavity: apply the paste on the bottom of cavity, close cavity securely.

Leave the filling in the cavity or in the root canal for 1 week period up to 30 days.

Product intended for multiple use except the applicator which is for single use only.

Multiple use of applicator may cause the risk of reinfection.

CONTRAINDICATIONS

Do not use CALCIPAST for patients with hypersensitivity to the product's components.

CAUTIONS

It acts irritating to the mucous membranes of the oral cavity. In case of contact rinse immediately with copious amount of water. It causes serious eyes injuries. In case of contact rinse immediately with copious amount of water and seek medical advice. In case of consumption do not extract emesis. Drink copious amount of water. Seek medical advice.

Necessary use of rubber-dam while working with the product. Use gloves, protective glasses and protective clothing.

Necessary use of rubber-dam while working with the product. Use gloves, protective glasses and protective clothing.

STORAGE

Store in original packaging at temperature below 25 °C.

Store out of reach of children.

Expiry date is placed on the main container.

Once container is open, the expiry date doesn't change if it is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly.

PACKAGING

Syringe of 2,1g of preparation, disposable applicators.



PPH CERKAMED Wojciech Pawłowski
37-450 Stalowa Wola, Polska
ul. Kwiatkowskiego 1
Phone/fax 0048 15 842-35-85

Last update: 18.04.2012