

# CANAL DETECTOR

Root canal orifices indicator

CE 2409

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY  
PRODUCT FOR DENTAL USE ONLY

COMPOSITION: methylene blue, propylene glycol, water Ph. Eur.

## INTENDED USE

Product CANAL DETECTOR is intended for use during endodontic treatment to identify the root canal orifices. Blue dye contained in the product invades into the root canal orifices and dyes them blue which makes it easier to find the main root canal orifices and detect the cracks.

## INSTRUCTIONS FOR USE

Before the treatment immerse attached applicator in spirit (ethyl alcohol) and then place it onto the syringe.

Remove dentine where the root canal orifices are expected, insert 1 – 3 drops of preparation into the bottom of the tooth cavity, wait 5 – 10 seconds, next rinse and dry the tooth cavity.

Points intensive dyed in blue indicate the root canal orifices.

After use withdraw the syringe piston to prevent overflow of preparation, screw off applicator and then screw the white plug into the syringe.

Product intended for multiple use except the applicator which is for single use only.

Multiple use of applicator may cause the risk of reinfection.

## CAUTIONS

Use rubber-dam while working with the product. Use disposal gloves, protective glasses and clothing.

## CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product's components.

## STORAGE

Store in original packaging at temperature below 25 °C.

Store out of reach of children.

Expiry date is placed on the main container.

Once container is open, the expiry date doesn't change if it is securely closed after each use.

## HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly.

## PACKAGING

Syringe of 2ml of preparation, disposal brush applicators .



PPH CERKAMED Wojciech Pawłowski  
37-450 Stalowa Wola, Polska  
ul. Kwiatkowskiego 1  
Phone/fax 00 48 15 842 35 85